

ColdGuard[®]

INSTRUCTIONS FOR USE

ColdZyme[®]

DEVICE DESCRIPTION

ColdGuard is an oral spray against common cold. It is easy to use and has fast onset of action. ColdGuard creates a protective barrier against common cold viruses by coating the mucous membrane in the oral cavity. The barrier traps the common cold virus, deactivates and inhibits them from spreading and multiplying.

INTENDED USE

To treat and alleviate common cold.

PRECAUTIONS

- Do not use ColdGuard if you are hypersensitive/allergic to any of the ingredients.
- Do not use the product during pregnancy or breastfeeding, as there is no clinical information available.
- Do not use the product after surgical intervention in mouth and/or throat until wound is healed, without consulting a healthcare professional.
- Do not inhale when applying the spray as it may cause irritation of the airways, resulting in cough and hoarseness.
- Keep out of reach of children, to prevent accidents with small parts.
- Consult a healthcare professional if you have a high fever or if your symptoms are more severe than those of a normal common cold; they could be due to something else.
- To avoid spread of disease, the product should only be used by one user.
- For optimal effect, do not eat or drink within 20 minutes after use.
- Do not use if package is damaged or opened.

INDICATION

To be used when exposed to common cold viruses or early on when common cold symptoms occur. ColdGuard is suitable for use in adults and children from 4 years.

CONTRAINDICATIONS

No known contraindications.

POSSIBLE SIDE EFFECTS

No known side effects.

Should you experience untoward effects while using ColdGuard, please consult your healthcare professional for advice.

CLINICAL BENEFITS

- ColdGuard protects against common cold viruses.
- ColdGuard can help shorten the duration of a common cold if used at an early stage of the infection.
- ColdGuard alleviates common cold symptoms and soothes sore throat.

DIRECTIONS FOR USE

When to use ColdGuard:

Start using ColdGuard as soon as possible after noticing symptoms of common cold or at possible exposure to common cold viruses. Continue to use ColdGuard until the symptoms are relieved or the exposure ceases.

How to use ColdGuard:

1. Open your mouth and aim the nozzle towards your throat (Figure 1).
2. Press down on the pump and spray 2 times (1 dose) every second hour up to 6 times daily during the time you are exposed to the cold virus or until the symptoms are relieved.

Figure 1




HOW DOES COLDGUARD WORK

ColdGuard consists of two major ingredients, glycerol and cod trypsin, which together provide an effective barrier that acts in three ways:


Traps: Glycerol creates an osmotically active transient barrier that traps viruses.

Deactivates: Trypsin inhibits the viruses' ability to bond, deactivating their ability to infect cells.

Protects: The ColdGuard barrier protects the oral cavity and throat.

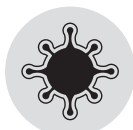

TRAPS

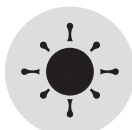

DEACTIVATES



PROTECTS

COLDGUARD IS AVAILABLE IN TWO DIFFERENT SIZES

- ColdGuard® 20 ml
- ColdGuard® 7 ml

All pack sizes may not necessarily be marketed at one time.

THE COLDGUARD PACK CONTAINS

- Oral Spray consisting of a bottle, pump, nozzle and protective cap.
- ColdGuard solution contains glycerol, water, a natural enzyme - cod trypsin, ethanol (<1 %), calcium chloride, trometamol and menthol. The solution is sugar-free and contains no preservatives, gluten or lactose.
- This instructions for use.

CARE AND STORAGE

Keep out of reach of children.

Store at or below 25 °C.

Use before the expiration date indicated on the product label and package, even if the product has not been opened.

Return all unused product to your pharmacist.

Do not dispose of unused product in drains or sewerage systems (e.g. toilets).

EXPLANATION OF SYMBOLS



Consult instructions for use



25 °C Upper limit of temperature



Use by date



Batch code



Catalogue number



Manufacturer



Caution, consult instructions for use



Do not use if package is damaged

DATE OF LAST REVISION OF INSTRUCTIONS FOR USE

March 2021

DISTRIBUTED AND MARKETED BY:

Abex Pharmaceutica (Pty) Ltd
Suite C, Rubenstein Ridge
617 Rubenstein Drive
Moreleta Park, 0181
South Africa
Tel: +27 12 997 6974
www.abexpharm.com

MANUFACTURED BY:

Enzymatica AB
IDEON Science Park
SE-223 70 Lund, Sweden
ColdZyme® is the registered trade mark of Enzymatica AB.



101007 ColdZyme® 20 ml
101108 ColdZyme® 7 ml

Medical Device



COL/IFU/C
L001ZA-03

0477