

ColdGuard®

INSTRUCTIONS FOR USE

ColdZyme®

DEVICE DESCRIPTION

ColdGuard is an oral spray that protects against upper respiratory tract viruses causing common cold and flu-like symptoms. ColdGuard is easy to use and acts fast. ColdGuard forms a protective, moisturizing barrier on the mucous membrane of the oral cavity and throat. The barrier disables viruses and inhibits their ability to spread and multiply.

INTENDED USE

Treat and alleviate common cold and flu-like symptoms.

PRECAUTIONS

- Do not use ColdGuard if you are hypersensitive/allergic to any of the ingredients.
- Do not use ColdGuard after surgical intervention in mouth and/or throat until the wound is healed, without consulting a healthcare professional.
- Do not inhale when applying the spray as it may cause irritation of the airways, such as cough and hoarseness.
- Keep ColdGuard out of reach of children, to prevent accidents with small parts.
- Consult a healthcare professional if you have a high fever or if your symptoms are more severe than those of a normal common cold; as the symptoms could be due to something else.
- To avoid spread of infectious disease, the product should only be used by one person.
- For optimal effect, do not eat or drink within 20 minutes after use.
- Do not use if package is damaged or opened.

INDICATION

ColdGuard can be used when exposed to upper respiratory tract viruses causing common cold and flu-like symptoms, or when experiencing these symptoms. ColdGuard is suitable for adults and children from 4 years.

CONTRAINDICATIONS

No known contraindications.
There are no known concerns regarding the use of ColdGuard during pregnancy or breastfeeding, consult a healthcare professional before use.

POSSIBLE SIDE EFFECTS

No known side effects.
Should you experience untoward effects while using ColdGuard, please consult your healthcare professional for advice.
Serious incidents (i.e. serious deterioration of the user's state of health) should be reported to the distributor (see contact details below).

CLINICAL BENEFITS

- ColdGuard protects against viruses causing common cold/upper respiratory tract infections/flu-like symptoms.
- ColdGuard shortens the duration of common cold/upper respiratory tract infection/flu-like symptoms if used at an early stage of the infection.
- ColdGuard alleviates common cold/upper respiratory tract infection/flu-like symptoms, including sore throat.

DIRECTIONS FOR USE


When to use ColdGuard:

Use ColdGuard at possible exposure to upper respiratory tract viruses or as soon as possible after noticing early symptoms of upper respiratory tract infection, such as sore scratchy throat or sneezing. Continue to use ColdGuard until exposure ceases or the symptoms are relieved.

How to use ColdGuard:

1. Open your mouth and aim the nozzle towards the back of your throat (Figure 1).
2. Press down on the pump and spray 2 times (1 dose) every second hour, recommended 6 doses daily during the time you are exposed to the upper respiratory tract viruses or until the symptoms are relieved. ColdGuard can be used for a longer period of time as there is no risk of overdose.

Figure 1

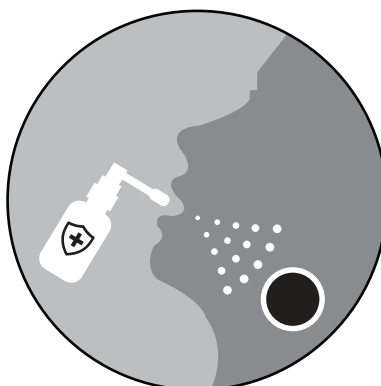

HOW DOES COLDGUARD WORK

ColdGuard forms an immediate barrier in the oral cavity and throat with triple action:

Catches: The physical barrier coats the oral cavity and throat, capturing viruses.

Disables: The barrier inhibits the captured viruses' ability to infect cells and multiply.

Protects: The physical barrier coats, moisturizes and protects the oral cavity and throat.



Catches



Disables



Protects

COLDGUARD IS AVAILABLE IN TWO DIFFERENT SIZES

- ColdGuard® 20 ml
- ColdGuard® 7 ml

All pack sizes may not necessarily be marketed at one time.

THE COLDGUARD PACK CONTAINS

- Oral Spray consisting of a bottle, pump, nozzle and protective cap.
- ColdGuard solution contains glycerol (55 %), water, a natural enzyme - cod trypsin, ethanol (<0,8 %), calcium chloride, trometamol and menthol. The solution is sugar-free and contains no preservatives, gluten or lactose. A small spring in the pump may contain traces of cobalt as being an impurity generally present in stainless steel.
- This instructions for use.

CARE AND STORAGE

Keep out of reach of children.

Store at or below 25 °C.

Use before the expiration date indicated on the product label and package, even if the product has not been opened.

Return all unused product to your pharmacy.

Do not dispose of unused product in drains or sewerage systems (e.g. toilets).

EXPLANATION OF SYMBOLS



Consult instructions for use



25 °C Upper limit of temperature



Use by date



Batch code



Catalogue number



Manufacturer



Distributor



Do not use if package is damaged



Medical device



Unique device identifier

DATE OF LAST REVISION OF INSTRUCTIONS FOR USE

October 2024

DISTRIBUTED AND MARKETED BY:



Abex Pharmaceutica (Pty) Ltd
Suite C, Rubenstein Ridge
617 Rubenstein Drive
Moreleta Park, 0181
South Africa
Tel: +27 12 997 6974

MANUFACTURED BY:



Enzymatica AB
IDEON Science Park
SE-223 70 Lund, Sweden

ColdZyme® is a registered trademark of Enzymatica AB, Sweden.



ColdGuard® 20 ml: 101007
ColdGuard® 7 ml: 101108



ColdGuard® 20 ml: 07340031105095
ColdGuard® 7 ml: 07340031106092



Medical Device

0477

COL/IFU/D
L200ZA-01